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GB

Autosana

Foam compression bandage for phlebological and orthopedic use
nonslip, permanently elastic, permeable to air

Constitution

Highly refined fine-pore polyurethane foam (not foam rubber) with a high proportion (up to 80%) of open pores, three-dimensional resilience and a high tensile strength.

Indications

Compressions after varix obliterations
Varicosis, phlebitis, thrombophlebitis and postthrombotic syndrome
Venous and lymphatic congestion
Tendovaginitis
Injuries of the ligaments, articular capsules and meniscus, heel joints and other postinjury states
Thermal therapy in arthroses

Properties

The Autosana bandage is an autonomous form of therapy. As such, it is suitable for compression treatment after varix obliterations. In addition, the Autosana bandage can be employed as an elastic supporting, relief or compression bandage. It fits closely to all uneven surfaces, i.e. also at the points of origin of the perforantes, since it is elastic in three dimensions and in addition clings securely to the skin because of its finely roughened surface.

- compression bandage with firm resilience
- great resistance to pressure and tensile strength
- nonslipping on conical limbs
- permeable to air and moisture
- washable (hand cleanser) and thus reusable
- good elastic restoration
- neutral to intact and damaged skin
- resistant to fats, oils, ointments, sweat as well as cleansers.

Application

In general, the 10 cm wide bandage is sufficient on the lower leg or arm, whereas the 12 cm wide bandage is recommended for the knee and upper leg.

- 1 Place the first turn of the bandage in such a way that the arch of the foot is raised and supported.
- 2 The number of circular turns depends on the degree of congestion and the constitution of the plantar arch.
- 3 Only release the heel when this has already been bound in a cloth bandage. Otherwise bandage the ankle.
- 4 The first ascending turn is placed over the beginning of the bandage.

- 5 The following turn covers half of the previous turn. Place the turns closer together only at the transition to a highly developed calf.

- 6 The bandage is continued up to the knee. The upper corner of the end of the bandage is pushed under the last turn.
Bandage the left leg with the right hand, and the right leg with the left hand.

Instructions for washing

Soak briefly in one of the conventional mild washing agents with lukewarm water, then wring well (do not brush) like a sponge, rinse well and dry in the air.
If this is considered necessary, bandages impregnated with secretion may also be boiled for ten minutes in addition without stirring or pounding, then rinse in lukewarm water, press out well and dry in the air. If necessary, a single or double steam sterilization is possible.

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