D

AUTOSANA

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GB

Autosana

Foam compression bandage for (phlebological and orthopedic use)

Characteristics

Highly resistant fibre-reinforced polyurethane foam (not foam rubber) with a high proportion (up to 60%) of open pores, three-dimensional resilience and a high tensile strength.

Indications

Compression after varicoid conditions
Varicosis, phlebitis, thrombophlebitis and postthrombotic syndrome
Venous and lymphatic congestion
Tendinitis

Injuries of the ligaments, articular capsules and meniscus, fall joints and other postinjury states
Thermal therapy in arthroses

Properties

- great resistance to pressure and tensile strength
- nonslipping on conical limbs
- permeable to air and moisture
- washable (hand cleanser) and thus reusable
- good elastic restoration
- neutral to intact and damaged skin
- resistant to fats, oils, ointments, sweat as well as deodorants.

Application

In general, the 10 cm wide bandage is sufficient on the lower leg or arm, whereas the 12 cm wide bandage is recommended for the knee and upper leg.

- 1 Place the first turn of the bandage in such a way that the arch of the foot is raised and supported.
2 The number of circular turns depends on the degree of congestion and the constitution of the patient's arch.
3 Only release the heel when this has already been bound in a cloth bandage. Otherwise bandage the arch.
4 The first ascending turn is placed over the beginning of the bandage.

- The following turn covers half of the previous turn. Place the turn closer together only at the transition to a highly developed calf.
The bandage is continued up to the knee. The upper corner of the end of the bandage is pushed under the heel turn.
Bandage the left leg with the right hand, and the right leg with the left hand.

Instructions for washing

Soak briefly in one of the conventional mild washing agents with lukewarm water, then wring well (do not brush) like a sponge, rinse well and dry in the air. If it is so considered necessary, bandages impregnated with secretion may also be soaked for ten minutes in addition without wringing or pouring, then rinse in lukewarm water, press out well and dry in the air. If necessary, a single or double steam sterilization is possible.

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NYCOMED Austria GmbH, Linz
GlobePharm AG
Emel Tic. ve Paz. Ltd. Şti.