Saibentüll, hydrophob
 Tulle gras vaseliné, hydrophobe
 Paraffin Gauze Dressing, hydrophobic
 Tul vaseline, hidrófobo
 Compressa de gaze vaselinada, hidrófoba
 Tulle diorepellente da guanto
 Saibentuell, hydrofoob
 Salvetyl, hydrofob
 Salvkompres, vattenavvisande
 Tyl s masti, hydrofobni
 Tyl s mastov, hydrofobny
 Kenőcsőtű, víztaázó
 Мазевая повязка, гидрофобная
 Tül s masčic, hydrofobovoy
 凡士林紗布敷料, 防水性

[illegible][illegible]

Composition:
Krotoned, white, non-latex glove made from 100 % cotton, impregnated with a hydrophobic water-resistant oilproof basis liquid (IPB).

Indications:
This product is used for the treatment of skin donor sites, leg ulcers, decubitus ulcers, burns, cuts, abrasions.

Contraindications:
Contraindications have not been observed.

Side effects:
Side effects have not been observed.

Interactions with other preparations:
Interactions have not been observed.

Dosage and instructions for use:
Prior to application, the wound should be cleaned as usual.
Immediately after it has been applied overlapping the wound edges by approximately 1 cm. With gentle pressure, a high adhesion dressing can be applied over the site (e.g. Vaseline®). The moisture does so slightly accelerating healing by means of a nonwoven dressing (e.g. Gazel® or Mepitel®) and thus preventing infection. The moist warm environment also helps to protect the wound against agents and bacteria from the surrounding area. The dressing can be removed at any time (e.g. Supraplast® IPB). On infected wounds, only if permeable dressings relevant material should be changed.

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GmbH & Co. KG · D-56579 Rengsdorf