# Kolonpassagezeit

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# Colonic Transit Time

## Composition

Six cellulose capsules, each containing 10 radio-opaque poly-urethane markers containing 40% barium sulphate. The capsules are numbered and contain markers of the following shapes:

## Capsule No. Capsule content

- Capsule 1: 10 rods
- Capsule 2: 10 spheres
- Capsule 3: 10 large rings
- Capsule 4: 10 cubes
- Capsule 5: 10 small rings
- Capsule 6: 10 rods

## Description

This test is a diagnostic aid which has been developed in order to investigate the passage of the contents of the colon and the rhythm of elimination. It comprises 6 capsules, which should be swallowed in the numbered sequence and at the given times. Each capsule contains 10 radio-opaque markers which do not undergo any changes in the gastro-intestinal tract. Without any special preparation of the patient, by means of a single X-ray picture (100KV) after 7 days (144 hrs) it is possible to reconstruct the activity of the colon during this period. The calculation is made according to Metcalf et al. (Gastroenterology 1987, 92 40-47) and Chaussade (Gastronétrol. clin. biol. 1986, 10: 385-389):  $COLONIC\ TRANSIT\ TIME = (Sum\ of\ the\ markers\ x\ [time\ between\ ingestions\ /\ number\ of\ the\ markers\ per\ capsule])$ : **CTT (hours) = sum of markers x 2.4**

## Determination of the colonic transit time CTT

Number of markers in the colon	<=10	11-20	21-30	31-40	41-50	51-
Corresponds to passage time in hours	<=24	26-48	50-72	74-96	98-120	122-
Normal values WOMEN (hours)			<=70			
Normal values MEN (hours)			<=60			

## Ingestion

- Days 1-6: 1 capsule at 9.00 a.m., with breakfast
- Day 7: Plain abdominal radiograph at 9.00 a.m.

## Instructions for use:

Prescription and explanation of use by the attending specialist.

#### Contraindications:

In case of known hypersensitivity to hydroxypropylmethylcellulose (HPMC / E464), barium sulphate BaSO<sub>4</sub>, Pellethane® 2363 TPU.

#### Warnings:

Not for use in pregnant women due to radiation concerns.

#### Symbols used:

	CE marking
	Manufacturer and date of manufacture
	Usable until
	Batch number
	Reference number
	Do not use if packaging is damaged
	Store at room temperature
	Follow instructions for use
	EU authorised representative

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