Sanyrene

Dispositif medical • Medical device • Dispositivo medico

fr **COMPOSITION ET PRESENTATION**

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URGO
MEDICAL

en **COMPOSITION AND PRESENTATION**

SANYRENE is a skin care oil rich in fatty acid glycerides (maize oil: 99%) and containing an anise perfume (1%). SANYRENE is presented in a spray bottle.

INDICATIONS

SANYRENE is intended to be used locally for :

- The prevention of pelvic pressure injuries in patients at risk of pressure injuries,
- The prevention of skin wounds induced by prolonged wearing of protective masks or goggles in health professionals or people in contact with the public.

Target population

Adults (including but not limited to the elderly)

PROPERTIES

- SANYRENE makes the skin more supple (emollient effect).
- SANYRENE helps maintain skin oxygenation when the skin is pressed.

CONTRAINDICATIONS

- In the absence of specific data, SANYRENE is not indicated in pregnant women, in breastfeeding women, or in children.
- Do not apply to a wound or to a mucous membrane.
- Do not use in case of known sensitivity to the product or to an ingredient included in its composition.

POTENTIAL ADVERSE SIDE EFFECTS

Consult a healthcare professional in the event of skin reactions (for example, redness, irritation), eczema or other allergic-type reactions (for example, itching).

Any signs or symptoms occurring during treatment with SANYRENE should be reported after a healthcare professional.

Any serious incident that occurs in relation to SANYRENE should be reported to the manufacturer (Laboratoires URGO) and the national competent authority.

PRECAUTIONS FOR USE

- It is recommended not to use the same bottle with different patients to avoid microbial contamination between patients.
- Handle the patient carefully.
- The product is oily: wait a few minutes before applying a preventive dressing to the same skin area, or before wearing a mask or goggles, and make sure that these devices are properly hold in place.

WARNINGS

- Keep out of the reach of children.
- Do not spray to or near a flame or incandescent object (for example, a cigarette).
- Do not use a flame or any incandescent object (for example, a cigarette) near clothing, bed linen, or a mask as these can catch fire if impregnated with oil.
- Do not use if the spray bottle is damaged.
- Do not use beyond expiration date.
- External use only. Do not swallow. If swallowed, rinse mouth immediately; do not induce vomiting. Ingestion may have a laxative effect.
- In case of contact with eyes or mouth, rinse immediately with plenty of water.
- Do not massage or rub the skin in order to avoid any skin damage, especially in patients at risk of pressure injuries.
- Talk to your doctor or other healthcare professional if you have a persistent skin redness or unusual appearance of the skin, as it may be a subcutaneous injury, an allergy, or another skin condition.
- If there is a pressure ulcer or a skin wound, stop applying the product and manage the wound.
- In case of fungal infection in the skin area at risk of pressure injuries, treat it first while at the same time applying the recommended prevention measures other than SANYRENE.

DIRECTIONS FOR USE

For external use only on the skin.

For the prevention of pressure injuries

2.Make a single spray on the pelvic skin area (trochanter, ischium and/or sacrum).

3.Spread the oil gently with your fingertips for about a minute, without massaging or rubbing in order to avoid any skin damage.

The local use of SANYRENE must be carried out within a global management of the patient including, for example, off-loading, frequent repositioning and body mobilization, repeated full skin inspections, protection against excessive moisture, incontinence management, and nutrition.

For the prevention of skin wounds induced by prolonged wearing of protective masks or goggles

1.Wash hands before and after use.

2.Make a single spray on your fingers.

3.Spread the oil gently on the face skin areas (on the nose, cheeks, especially at the cheekbones, around the ears, and/or on the forehead) with your fingertips for about a minute, without massaging or rubbing in order to avoid any skin damage.

Frequency of application

Apply SANYRENE two to four times a day, especially when changing position or before wearing a new mask.

Regularity and frequency of application are important to help prevent pressure injuries or skin wounds induced by prolonged wearing of protective masks or goggles.

To discard the bottle

In community care, discard it with household waste. In hospital, dispose of according to the procedures in force.

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