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PZN 0761621

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Contragel® green

Application instructions (Please keep these instructions!)

To be used with natural latex rubber or silicone diaphragms or cervical caps (contraception). Also compatible with natural latex rubber or polyurethane condoms. Water soluble, limpid.

Application

Before inserting the diaphragm or cervical cap put 3 ml (a spoon full) of Contragel Green into the diaphragm or cervical cap. Then follow the instructions for insertion of the diaphragm or cervical cap.

Composition

Water, Lactic acid, Sodium lactate, Cellulose, Sorbic acid.

Storage

Store in a dry place between 4°C and 25°. Always keep closed. Manufacturing and Expiry Dates see box bottom and tube print.

Tube to be used within 3 months after opening.

Side Effects

Contragel Green has no known side effect on the pH value of vaginal flora. It can be used as often and as long as required. In case of the occurrence of any problem by using Contragel Green, please stop using it and ask your pharmacist, health adviser, or family planning clinic.

Contraindications

There is no information available about women using Contragel Green experiencing adverse side effects.

Information:

Contragel Green is only effective when used together with a diaphragm (e.g. Caya Diaphragm) or a cervical cap (e.g. FemCap or Lea Contraceptivum). Contragel Green as a contraceptive gel is **not** a safe contraceptive method when used **alone**, i.e. without a mechanical barrier.

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DeltaMed GmbH
Kessel GmbH, 64546 Mörfelden/Walldorf, Germany
Kessel GmbH, 64546 Mörfelden/Walldorf, Duitsland