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Made in the UK	
Packaged and Sterilized in Canada	
	
Derma Sciences, Inc. 104 Shorting Road Toronto, Ontario M1S 3S4 USA Int: +1 609 514 4744 Canada Int: +1 416 299 4003	
Derma Sciences Europe Ltd. P.O. Box 220 5 High Street Maidenhead SL6 1JN UK Int: +44 1628 779 461 Freephone (UK only): 0800 071 3912	
www.dermasciences.com	
	

DERMA SCIENCES, INC.
 ALGICELL® Ag dressing can be applied for up to five days. However, the dressing change frequency will depend on the condition of the patient as well as the level of wound exudate.
 Reapply when the secondary dressing has reached its absorbent capacity or as directed by a wound care professional. Moisten with sterile saline if the wound bed appears dry before removal of the dressing. Remove the dressing from the wound bed gently. Cleanse the wound bed prior to application of new dressing.
CAUTION: Do not use product if the immediate product packaging is damaged.
Store at room temperature: 10 – 25°C (50 – 77°F).
Keep dry.
This product is intended for single use and re-use of this product will not provide sufficient antimicrobial activity. Sufficient data & experience not available for the use of this product in neonates and small babies.

 ALGICELL® Ag dressing can be applied for up to five days. However, the dressing change frequency will depend on the condition of the patient as well as the level of wound exudate.
 Reapply when the secondary dressing has reached its absorbent capacity or as directed by a wound care professional. Moisten with sterile saline if the wound bed appears dry before removal of the dressing. Remove the dressing from the wound bed gently. Cleanse the wound bed prior to application of new dressing.
CAUTION: Do not use product if the immediate product packaging is damaged.
Store at room temperature: 10 – 25°C (50 – 77°F).
Keep dry.
This product is intended for single use and re-use of this product will not provide sufficient antimicrobial activity. Sufficient data & experience not available for the use of this product in neonates and small babies.

ALGICELL® Ag is a registered trademark of Derma Sciences, Inc.
©2010 Derma Sciences, Inc.

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