## 3M™ Cavilon™ Durable Barrier Cream en

### Product Description

3M™ Cavilon™ Durable Barrier Cream is a pH balanced, hypoallergenic, concentrated cream that provides skin protection from body fluids. The cream protects skin by providing a long lasting barrier. The cream also moisturises and conditions skin. Use of this cream allows adhesive products to stick to the skin.

### Indications for Use

Protection of at-risk skin from damage associated with incontinence and related symptoms such as anal itch. It may also be used to moisturise and condition dry skin. This product is safe to use on both intact and injured skin for example associated with incontinence.

### Ingredients:

Water (Aqua), Dimethylol Adipate, Acrylate Terpolymer, Cocos Nucifera (Coconut) Oil, PPG-15 Stearyl Ether, Glycerin, Isopropyl Palmitate, White Mineral Oil (Paraffinum Liquidum), Trimethylsilyloxyoctyl, Paraffin, Dimethicone, Phenoxethanol, Benzoic Acid, Dehydroacetic Acid, Magnesium Sulfate.

### Warnings

FOR EXTERNAL USE ONLY. Avoid contact with eyes.

If condition worsens or does not improve within 7 days, consult a physician. Keep out of reach of children. In cases of ingestion, seek medical help. Avoid use on persons with known allergy or sensitivity to any ingredient listed. Cavilon Durable Barrier Cream is not intended to treat anal disease.

### Precautions

Cavilon Durable Barrier Cream may increase the adherence of some adhesive products.

**Directions for Use**

Gently cleanse skin as required. Apply Cavilon Durable Barrier Cream sparingly to cover entire affected area.

Repeat as necessary. If the feet or eye, too much has been applied.

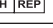
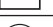
**Storage/ Shelf Life/Disposal**
For best results, store at room temperature. Do not freeze. Avoid excessive heat and humidity.

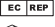
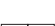
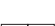
For shelf life, refer to the expiration date on each package.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

For further information please contact your local 3M representative or contact us at 3M.com and select your country.

### Explanation of Symbols

Symbol Title	Symbol	Description and Reference
Authorized Representative in European Community		Indicates the authorized representative in the European Community. ISO 15223, 5.1.2
Batch code		Indicates the manufacturer's batch code so that the batch or lot can be identified. ISO 15223, 5.1.5
Catalogue number		Indicates the manufacturer's catalogue number so that the medical device can be identified. ISO 15223, 5.1.6
Consult instructions for use or consult electronic instructions		Indicates the need for the user to consult the instructions for use. Source: ISO 15223, 5.4.3
Importer		Indicates the entity importing the medical device into the locale. Source: ISO 15223, 5.1.8
Swiss Authorised Representative		Indicates the authorised representative in Switzerland. Source: Swissmedic.ch
CE Mark		Indicates conformity to European Union Medical Device Regulation or Directive.
Date of Manufacture		Indicates the date when the medical device was manufactured. Source: ISO 15223, 5.1.3
Do not re-use		Indicates a medical device that is intended for one use or for use on a single patient during a single procedure. Source: ISO 15223, 5.4.2
Fragrance free		Indicates medical device does not contain a fragrance additive
Green Dot		Indicates a financial contribution to national packaging recovery company per European Directive No. 94/62 and corresponding national law. Packaging Recovery Organization Europe.
Manufacturer		Indicates the medical device manufacturer as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC. Source: ISO 15223, 5.1.1
Medical Device		Indicates the item is a medical device.
Unique device identifier		Indicates a carrier that contains unique device identifier information. Source: ISO 15223, 5.7.10
Does not contain natural rubber latex		Indicates there is no presence of dry natural rubber nor natural rubber latex as a material of construction within the medical device or the packaging of a medical device. Source: ISO 15223, 5.4.5 and Annex B
Parabens free		Indicates medical device does not contain a paraben additive.
Use-by date		Indicates the date after which the medical device is not to be used. Source: ISO 15223, 5.1.4

			<b>3M Deutschland GmbH</b> Health Care Business carr-schutz-str. 1 41463 Neuss Germany
			<b>3MSchweiz GmbH</b> Eggstrasse 91 8803 Rüschlikon Switzerland
			Update date: 2023-03-31 3M and Cavilon are trademarks of 3M. © 2023, 3M. All rights reserved. 0233475.2

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