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EN **Cleansing and hygiene oil for the external acoustic meatus**
Auricular Spray - No gas

20 ml - 0.70 FL. OZ. Bottle

COMPOSITION

Tocopherol acetate, hydrogenated polydecene

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EXCLUSIVE SALES LICENCEE FOR ITALY

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WHEN SHOULD FILME OTO BE USED

FILME OTO is an oil based tocopherol acetate (vitamin E), which promotes delicate and thorough cleansing of the external auditory canal, while respecting its physiological equilibrium. This unique composition is similar to the skin lipids, and helps thin and loosen earwax for easy removal, allowing natural discharge from the ear canal without the use of "cotton swabs". FILME OTO is indicated for the prevention and removal of ear wax plugs in children and adults. It is also used in the treatment of chronic eczema of the auditory duct and in subjects wearing hearing aids and head phones for prolonged periods.

FREQUENCY AND WHEN THE PRODUCT SHOULD BE ADMINISTERED

Prevention of ear wax build up: 1 squirt in each ear 3 times a week, at least 30 minutes before taking a shower or bath.

To remove ear wax build up: 1 squirt 3 times a day, at least 5 days before removing the wax.

In case of Chronic Eczema of the acoustic meatus apply 1 squirt in each ear once a day for a month.

In subjects wearing external hearing aids apply FILME OTO in the ear canal without prosthesis. The prosthesis can be reinserted 3 hours after using FILME OTO, remove any excess product before reinserting.

CONTRAINDICATIONS

Hypersensitive towards the components.

In case of ear infection or perforation of the eardrum consult a doctor before use.

WARNING

KEEP OUT OF REACH OF CHILDREN.

INTERACTION WITH MEDICINES AND OTHER INTERACTIONS

There are no known interactions with other medicines or products used on ear canal. In the case of application of more than one product leave an interval of at least 5 minutes between each product. FILME OTO should be applied last.

HOW TO USE THIS DEVICE

1. Position the nozzle perpendicular to the bottle
2. Before using the first time, pump by depressing 5 times
3. Tilt your head to one side and introduce the dispenser at the entrance of the ear cavity
4. Press the dispenser
5. Wait for a few seconds, lift your head back to position and wipe off any excess
6. If necessary, repeat the process in the other ear
7. For prevention leave for at least 30 minutes and then wash normally (bath / shower)
In case of Cerumen Impaction (Earwax Buildup and Blockage) use before following earwax medical procedure for removal

METHOD OF ADMINISTERING

For topical ear use.

SIDE EFFECTS

Although FILME OTO is extremely tolerable, in some rare cases it may cause a stinging effect, irritation, allergic reaction or reddening and swelling at the site of application. If this occurs discontinue use and consult a doctor. If any of the side effects gets serious, or if you notice any side effects not listed in this leaflet, please tell your doctor or pharmacist.

STORAGE OF THE DEVICE

DO NOT USE AFTER THE INDICATED EXPIRY DATE ON THE INNER AND OUTER PACKAGING.

STORE AT ROOM TEMPERATURE.

Europ. Patent

MEDICAL DEVICE. DO NOT DISPERSE IN THE ENVIRONMENT.

AFTER USE, DISPOSE OF IN ACCORDANCE WITH YOUR NATIONAL LEGISLATION.

NO FRAGRANCE
NO COLOURANTS
NO PRESERVATIVES
NO ESSENTIAL OILS
Nickel (< 1 ppm) and Gluten (< 10 ppm) tested

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