MAVAMED

FUNGAL NAIL SOLUTION with brush

This product contains the following active ingredients: urea and lactic acid.

General advice - Carefully read the instructions for use prior to using MAVAMED Fungal Nail Solution, even if you have used this medical device before. The information may have been amended according to new research and findings since you used this product last. This medical device can be purchased without prescription. Nevertheless, MAVAMED Fungal Nail Solution should be used with great care in order to obtain the desired result. Keep these instructions for use in case you need to consult them again.

Symptoms - Onychomycosis is a nail infection caused by microscopic fungi. They proliferate in hot, dark and wet places (like shoes) and are fed with keratin, the main component of nails. Various symptoms can result from a nail infection: the nail can discolour (white, yellow, ...), can become thicker or brittle, or can even flake.

Transmission - The nail infection is contagious and some conditions can increase the risk of contamination: damaged nails, excessive sweating, diabetes, bad blood circulation, some sport practices like swimming, judo or running, etc.

Prevention - There are some precautions that can be taken to avoid being infected: wearing shoes with good aeration, preferring socks made out of wool or cotton to avoid humidity, drying feet thoroughly after a shower, sport or swimming, wearing sandals in wet public places like swimming pools and not wearing someone else's shoes.

Mode of action - MAVAMED Fungal Nail Solution is a medical device that treats fungal nail infections (Onychomycosis). The "Instant Killer" complex of this product stops in a few seconds the development of fungus, avoiding spreading onto other nails and brush.

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It acts as a physical barrier that prevents the spreading of mycosis. It is composed of:

- lactic acid that reduces nail pH thus creating an unfavourable environment to the development of mycosis and a blending of essential oils (lavender and eucalyptus);
- urea that allows the active ingredients to deeply penetrate into the nail plate.

This product is intended for external use, and for nail plate application only.

How to use - Before use, wash the affected area with water and soap and dry thoroughly. Brush product twice a day (morning and night) which will form a colourless film on the infected nail. Leave on to dry in the open air for a short time. To be more effective, cover your treated nail with a plaster before wearing socks or shoes.

It is important to continue using MAVAMED Fungal Nail Solution until the infection is totally cured and a healthy nail has grown. It can take several months. If you do not observe any improvement after 3 months of use, consult your doctor.

Contraindications - Do not use MAVAMED in the event of:

- oversensitivity to one or more of the ingredients;
- if you are in doubt about using MAVAMED Fungal Nail Solution and/or if you can use it in combination with other medication, then please consult your doctor or pharmacist.

Side effects - In exceptional cases, possible side effects may occur: erythema, irritation, edema, pain, nail discoloration and onycholysis. In case of any undesirable side effect, stop using MAVAMED Fungal Nail Solution and consult your doctor or pharmacist.

 Store at room temperature (15-25°C) in a dry and dark place in the original container.

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Warnings - External use only. Avoid contact with eyes or mucous membranes. Do not apply to open wounds or damaged skin. Do not bandage tightly. Do not apply heat to the area of application. Keep out of reach of children. Not suitable for children under 18 years old. Women who are pregnant or breastfeeding should not use this product. Do not use after the expiry date.

Classification - MAVAMED Fungal Nail Solution is a class IIa medical device according to the definition of the Council Directive 93/42/EEC.

For the expiry date please see bottom of packaging.
Date and version on which these instructions for use were last revised: 2020-12 v3

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