## Evolstin® Anti Warzen Stift

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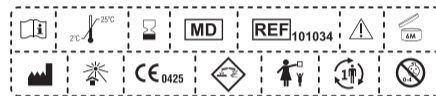
**CE0425**  
PZN 18359049

## Evolstin® Anti Wart Pen

 EVOLSTIN® ANTI WART PEN contains a corrosive agent.

 Read this instruction carefully prior to use.

 For external use only. Keep out of sight and reach of children.

**INDICATION:**

EVOLSTIN® ANTI WART PEN is indicated for the treatment of warts and verrucas found on the feet, hands, body, arms and legs. The product can be used by adults and children from 4 years old.

Warts and verrucas (hereafter defined as warts) are local growths in the skin caused by the human papilloma virus (HPV). They may be found alone or in clusters. There exist different types of warts. Most commonly found on hands, feet, elbows and knees. Warts are contagious and spread by contact of the virus with skin.

EVOLSTIN® ANTI WART PEN is a CE marked Medical Device. The pack comprises a treatment pen and this instruction leaflet.

EVOLSTIN® ANTI WART PEN contains 3 ml sufficient for minimum 220 applications. Evolstin anti wart pen is specifically designed for topical administration directly to warts.

EVOLSTIN® ANTI WART PEN is immediately absorbed into the wart. The pen contains a keratolytic compound, which breaks down and dries tissue locally and thereby eliminates warts.

**WARNINGS & PRECAUTIONS:**

- Only for external use. Keep out of reach of children. The product contains a corrosive agent with strong smell. Do not inhale.
- Do NOT use the product for genital warts or in the facial area.
- Do not use Evolstin anti wart pen if you are known to be hypersensitive or allergic to the ingredients as well as bee / ant stings.
- When applying avoid contact with healthy skin surrounding the wart.
- Do not expose the treated area to sunlight until the skin has returned to normal.
- Do not apply Evolstin anti wart pen to sore or bleeding warts, irritated, inflamed, blistered or damaged skin. Pause until the skin has recovered.
- Do not use on warts with hairs, birthmarks, dark moles, pigmented skin or in case you have doubts if the skin condition is a wart.
- Extra care should be taken when applying the treatment to thin and sensitive skin and when treating children and people with diabetes
- **It is better to apply too little liquid if in doubt. Over application can result in side effects such as wounds, blisters, inflammation, tissue death (necrosis), scars and chemical burns.**
- **Do not exceed the number of recommended applications.**
- Consult your doctor if the treated area becomes red and painful or gets inflamed.
- Consult your doctor if the treated area becomes red and painful or gets inflamed.
- Single patient - multiple use.
- Do not use in combination with other wart removal therapies.
- Do not use for children under 4 years of age, unless directed by the doctor

**SHOULD THE PRODUCT ACCIDENTALLY COME INTO CONTACT WITH THE EYES OR MUCOSAL MEMBRANES, RINSE WITH PLENTY OF WATER FOR 15 MINUTES AND SEEK MEDICAL ADVICE .**

**HOW TO USE EVOLSTIN® ANTI WART PEN:**

It is recommended to soak the wart in luke warm water for at least 5 minutes to soften the skin prior to treatment. If the wart is covered by hard skin gently scrape the surface of the wart (e.g. with a scissors blade) and carefully remove the hard skin. Do not file the wart. It is recommended that adults apply the treatment to children.

1. Remove the cap from EVOLSTIN® ANTI WART PEN
2. Gently massage the wart with the tip of the treatment pen for a maximum of 20 seconds. The solution penetrates immediately. Make sure to cover the whole wart and take care not to touch or spill onto surrounding healthy skin. Recap the pen firmly after application.
3. Repeat the treatment 3 times a week ( a day apart followed 2 days rest) for up to 6 weeks. Do not exceed the number of recommended applications.

**Do not exceed the number of recommended applications.**

The EVOLSTIN® ANTI WART PEN delivers a small amount of a colorless solution. If in doubt whether the tip of the pen is dry, press the tip against a piece of absorbent paper to check the liquid flow.

**DON'T SHAKE THE PEN.**

Should any discomfort or pain occur stop application. If it burn, sting or if serum is applied to healthy skin rinse with cold water.

**POSSIBLE SIDE EFFECTS:**

It is important to use the product as instructed. Do not overexpose as this can cause skin damages (please see PRECAUTIONS above).

A mild tingling or burning sensation can in some cases be felt for 10-15 minutes. If overexposed temporary side effects can occur such as stinging, redness, pain or bleeding. Pause treatment until the skin has fully recovered. There are no known risks for pregnant or breastfeeding women.

Serious incidents should be reported to the manufacturer and relevant authority.

**STORAGE:**

Store at 2-25°C degrees Celsius and out of direct sunlight enclosed in the original packaging and out of the reach of children. Do not use after the expiry date or 6 months after removing the cap from the pen for the first time. Carefully recap the pen after use.

**The product can be disposed xxx**

**INGREDIENTS:**

Formic acid (85%), Dimethyl isosorbide, Aqua, Caprylic acid, Origanum Vulgare leaf Extract. ICI11. 01garun VU\III\I leaf Extract.

**CONTENT:** 3 ml, sufficient for minimum 220 applications.

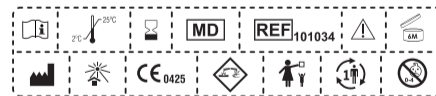
**REGISTRATION:** EVOLSTIN® ANTI WART PEN is classified and registered as Medical Device Class IIa.

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