front
170mm

(DE) GYNELLA® Balance

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Na Pankráci 14
140 00 Prag 4
Tschechische Republik

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(EN) GYNELLA® Balance

GYNELLA® Balance is a medical device based on non-viable tyndallized lactic bacteria (*Lactobacillus acidophilus*), hyaluronic acid, soy isoflavones and lactic acid. Thanks to its composition it maintains natural vaginal defences and hydration equilibrium when the balance of vaginal microbial flora may be temporarily altered by physiological conditions (hormonal changes, menopause, stress, pregnancy, postpartum damage, breastfeeding) and/or external factors (antibiotics, tampons, free time activities, incorrect hygiene, anti-conceptual devices).

Use/indication:

Typically, women in menopause, due to the drop in estrogen levels, experience vaginal dryness, itching, irritation, burning, dyspareunia. Indeed, vaginal dehydration, brought on by chemical or physical changes in the body, can lead to symptoms not only in response to sexual activity (dyspareunia) but also during simple activities such as walking (itching, unattractive odour, and discomfort). The uro-vaginal system is characterized by a complex microbial flora which physiologically acts as a natural defence against bacterial infections: the pH value of normal vaginal secretion is around 4.5 and this acidity is maintained by a normal level of estrogens and by a constant lactic flora. The lactic bacteria, through their metabolism, produce lactic acid which is responsible for maintaining an acid environment in the vaginal mucosa and this acidity prevents and protects against invasion and growth of pathogenic microflora. GYNELLA® Balance with its combination of soy isoflavones and hyaluronic acid hydrates the vaginal mucosa in menopausal women and together with tyndallized lactic ferment and lactic acid helps to maintain the vaginal microbial balance.

Instructions for use/dosage:

Unless otherwise prescribed, apply the device once a day for 10 days preferably in the evening before bedtime.

- Wash the intimate parts thoroughly (preferably with both external and internal vaginal lavage).
- Open the pack and after washing your hand well, take a single suppository by the blister.
- Insert gently into vagina trying to get deep, preferably in supine position.

Side effect:

Known hypersensitivity to any of the ingredients.

Storage:

Do not store the package above 25°C, keep tightly closed in a dry place, away from light and heat sources.

Warnings:

Keep out of reach and sight of children. Do not swallow. Do not use if

package as previously opened or damaged. Do not use the product after the expiry date shown on the package. The expiry date refers to the integral and properly stored product. Do not use the product in case of vaginal lesions or injuries. Do not use during menstruation. Do not exceed the recommended daily dose without medical advice. During pregnancy and breastfeeding the use must be limited only if strictly required and under the direct supervision of your doctor. Disposable, re-use may affect the safety of the device. Changes in suppository's aspect, like colour variation from pale white to brownish colour, does not affect the quality and efficacy of the product.

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Velit Biopharma S.r.l.