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CE 0481

spray will adhere well to the skin and, therefore, the active ingredients will be fixed and provide longer term efficacy at the right place.

**Possible side effect(s):** There are no reported serious side effects. Side effects could be skin irritation such as itching, swelling, rashes and redness.

**Use during pregnancy and/or breastfeeding:** No data is available about using Athlete's foot spray during pregnancy or the breastfeeding period. If you are pregnant or breastfeeding, please consult your doctor or pharmacist before using this product.

**Warnings:**

- Keep out of reach of young children;
- If you are allergic to one of the ingredients or experience an allergic reaction, stop using Athlete's foot spray;
- Only for external use;
- Avoid contact with eyes or mucous membranes;
- Consult your doctor when you do not see an improvement within two weeks.

**Content:** 25 ml

**Composition:** Alcohol Denat, Aqua, Glycerin, Urea, Natilact®, Phenoxyethanol, Ethylhexylglycerin, Camphor, Menthol, Lactic Acid, 4-Terpineol.

**Storage instructions:** Store at a temperature of 5-25°C in a dry and dark place in the original container and out of reach of young children.

For the expiry date please see the bottom of the packaging.

**Medical device:** Athlete's foot spray is a class IIa medical device in accordance with the Medical Device Directive 93/42/EEC.

Date on which these instructions for use were last revised:  
08-2020 • 0725/68-V1.0

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Medical Device

## Athlete's foot spray

- **To support the treatment of athlete's foot**
- **Hygienic- No hand contact required**
- **To reduce itchiness and irritation and prevent an unpleasant smell**

**Advice:** Carefully read the instructions for use before using Athlete's foot spray even if you used this medical device before. The information may have been amended according to new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. Nevertheless, Athlete's foot spray should be used with great care in order to get a good result. Keep these instructions for use in case you need to consult them again.

**Application:** Athlete's foot spray is intended to support the treatment and prevention of athlete's foot (ringworm of the foot or tinea pedis). Athlete's foot often goes hand-in-hand with an itchy red rash between the toes and on the soles of feet. Athlete's foot will also often lead to an unpleasant smell. Athlete's foot is infectious. The fungus spreads easily at hot and moist places such as, for example, swimming pools, showers and wet floors. Athlete's foot spray reduces itchiness and irritation and prevents the unpleasant smell.

**Method of application:** Dry both feet and apply the spray twice a day (in the morning and in the evening) as a thin layer on both feet. This is required because the fungus may also be present on the other foot without it being visible. Apply the spray under and in-between the toes and on the soles and sides of your feet. Allow the spray to dry for 1 or 2 minutes so that it can create the protective layer.

**Duration of use:** An improvement can usually be seen after a few days. Continue the treatment for a week after the athlete's foot has completely cleared.

Note: Irregular use or stopping the treatment prematurely will increase the probability that your symptoms will return.

**How it works:** Athlete's foot spray is a medical device to support the treatment of athlete's foot. Athlete's foot spray is a spray that creates a protective layer to the skin of the foot. The combination of ingredients penetrates the upper skin-layer and callus and provides an environment that is unfavourable for the development of fungus that causes athlete's foot. Natilact is a natural ester that protects and hydrates the skin. The

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Peter Bausch GmbH & Co. KG.