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Multifunctional solution for soft contact lenses

360 ml

Barnaux Healthcare B.V. 019EFM068403642
Vluchtoord 38 05/2023
NL-5406 XP Uden

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EN Multifunctional solution

Cleaning, removal of protein and lipid deposits, disinfecting, inserting and storage. For soft contact lenses.

Composition:

PHMB 0,0001%, POLOXAMER 407, POLOXAMER 188, SODIUM CHLORIDE, EDTA, PHOSPHATE BUFFER, PURIFIED WATER.

Before using, carefully read the instructions and warnings to ensure appropriate use of this medical device.

Precautions and instructions for use:

Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel. Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal. Rinse the lenses carefully with contact lens solution and put the lenses in the lens case. If rubbing is required due to protein or lipid deposits, place the lens in the palm of your hand and add a few drops contact lens solution. Rub the lens gently between the palm of your hand and your finger up to 30 seconds. Rinse the lenses thorough with the solution. Fill your lens case with contact lens solution and place your lenses in the lens case. To disinfect the lenses, store the lenses for at least 4 consecutive hours. After disinfecting, rinse both sides of the lens with contact lens solution. The contact lenses moisturized with contact lens solution can be inserted. Rinse the lens case with contact lens solution and allow the lens case to dry in the air. Repeat disinfection steps when the lenses are stored unused in the lens case for a period longer than 7 days. Do not touch the dropper tip of the bottle as this may contaminate the solution. Close the bottle tightly immediately after use. Replace your lens case frequently to prevent infection due to contamination.

Storage conditions:

Use within 4 months after opening. Keep out of reach of children. Keep the solution in its original packaging.

Warnings and contra-indications:

Do not use if you are allergic to one of the ingredients. Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor. If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution. Do not use when the packaging is damaged (See image 1). Do not use after the expiry date. Do not change your contact lens care system without consulting your contact lens specialist. Always replace any remaining solution in the lens case with fresh solution. Never use tap water as an alternative for contact lens solution. Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

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eye.CARE[®]
C O M P A N Y

eyecare company AG
Postfach 501405, D-50974 Köln
www.eyecare-company.de
500012_REV03_LAB_20251009

MD

STERILE A