DE

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Buffered isotonic sodium
chloride solution

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EN

Isotonic solution for inserting and rinsing.
For all types of contact lenses.

Before using, carefully read the instructions and warnings to ensure appropriate use of this medical device.

COMPOSITION:

SODIUM CHLORIDE, EDTA, POLYSORBATE 80, BORAX/BORIC ACID, PURIFIED WATER.

PRECAUTIONS AND INSTRUCTIONS FOR USE:

The rinsing solution can be used in combination with contact lens care solutions as part of your contact lens care regimen. Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel. Do not use when tamper-evident seal is broken before the first time use. Unscrewing the cap from the bottle will break the tamper-evident seal. Inserting: place the lens in the palm of your hand and add a few drops solution. Insert the lenses into your eyes. Rinsing: place the lens in the palm of your hand and rinse it with contact lens solution. Do not touch the dropper tip of the bottle as this may contaminate the solution. Close the bottle tightly immediately after use.

STORAGE CONDITIONS:

Use within 2 months after opening. Keep out of reach of children. Keep the solution in its original packaging.

WARNINGS AND CONTRA-INDICATIONS:

The rinsing solution is not an alternative for disinfection or storage solution. Always use a multipurpose contact lens solution or disinfection solution in combination. Do not use the contact lens solution if you are allergic to one of the ingredients. Stop wearing contact lenses when you experience eye irritation or when you have an eye infection and consult a doctor. If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lens solution. Do not use when the packaging is damaged (See image 1). Do not use after the expiry date. Do not change your contact lens care system without consulting your contact lens specialist. Always use fresh rinsing solution. Never use tap water as an alternative for contact lens solution. Stop using the product and inform the legal manufacturer and the national health authorities when you experience a serious health issue in relation with this medical device.

Ein Produkt der / a product of:

eye.care
COMPANY

eyecare company AG
Walther-Rathenau-Str. 5
50996 Köln, GERMANY
www.eyecare-company.de

4