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Instructions for use

GB

Tears Again® liposome spray with vitamins A + E

For the treatment, soothing or compensation of irritations in the dry eye.

Composition

1 ml contains 10 mg soy lecithin, 8 mg sodium chloride, 8 mg ethanol, 5 mg phenoxyethanol, 0,25 mg vitamin A, 0,02 mg vitamin E, aqua purificata

Areas of application

Tears Again® stabilises the lipid layer at the lacrimal film, regulates and improves the humidification of the eye surface and the eyelids. It should be used in environment related disorders as dry eyes, strained eyelids, foreign body sensation, and a burning or itching sensation of the eyes.

Contraindications

None known

Interactions with other drugs

None known

Directions for use

Spray Tears Again® 1–2 times on the closed eyelids. Do not spray the product into the open eye or touch the eye surface with the spray head. Keep a distance of approx. 10 cm. A slight short-term burning sensation may occur when it is used inadvertently in the open eyes. However, this feeling passes very quickly and causes no harm.

When using eye cosmetics, Tears Again® should be used before applying make-up and after make-up removal. In principle however, Tears Again® can also be used with previously applied eye make-up. In this case, only one spray should be applied and the distance between the dispenser and the eye should be slightly greater (about 20 cm).

Tears Again® makes wearing contact lenses much more comfortable.

General recommendation: use Tears Again® 3 to 4 times per day. For more severe cases, Tears Again® may be used more often without consideration.

The use of Tears Again® is not limited to a specific period of time.

Storage

Store in a cool and dry place.

Keep out of reach of children.

Once open, the bottle may be used until the last spray.

There is no time limit for open bottles.

Do not use after expiry date.

Notes

The bottle may also be used by several persons, since there is no contamination.

Tears Again® should not be used simultaneously with eye medicine. Should your doctor prescribe you eye drugs other than Tears Again®, you should keep an interval of ten minutes between the two applications.

Tears Again® should not be used in the presence of hypersensitivity to one of the ingredients.

It is important the seal (crimp seal) of the spray pump fits correctly and is undamaged before use.

The bottle is filled amply, since for technical reasons the bottle cannot be emptied completely. It is guaranteed, that 20 ml can be used.

Properties

Tears Again® is effective with its tiny liposomes. One spray applies millions of the small particles on the eyelid. They make the skin moist, soft and subtle, lower the temperature of the eyelid and soothe eye disorders that are caused by computer work, air conditions, draught, dry environment air and long car and air journeys.

Liposomes penetrate the skin quickly, and provide it actively with water, phospholipids with essential fatty acids, oil acids, linoleic and linolenic acids and vitamin E. They ease itchiness, have painrelieving and antiphlogistic properties, which support a quick relief of possible inflammations. The eyelids and their surroundings are cared for intensively, membrane disorders are balanced and the skin functionality remains intact.

When using for the first time, 3 to 4 pumping thrusts are required to fill the standpipe.

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